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DIN EN ISO 7376

(Anaesthetic and respiratory equipment – Laryngoscopes for tracheal intubation)

Instructions

ENGLISH

HEINE Standard F.O. LED / L LED Laryngoscope Rechargeable Handle

Read these instructions carefully before using the HEINE Standard F.O. LED / L LED Laryngoscope Rechargeable Handle and keep them in a safe place for future reference.

Safety Information

Symbols:

In accordance with local regulations this product should be disposed of as an electronic device separately.

Range of applications: HEINE Standard F.O.LED and L LED Laryngoscope rechargeable handles are designed for use with F.O. laryngoscopes for intubation only.

General safety:

- If you suspect any defect, do not use the handle.
- Do not open the illumination head of the battery insert (2) or the battery (3)(4) or the bottom cap (5)(6).
- Do not shine light directly into the eyes to avoid dazzle, especially if the pupils are dilated.
- All repairs other than a change of batteries should be carried out by qualified persons.

Safety in use:

- Use only blades and handles with a green mark together.
- Bring the blade into the normal working position and check that the light switches on.
- Do not use in the presence of strong magnetic fields.
- Charge rechargeable handles before use.
- The handles should not be used in the presence of explosive substances.
- Rechargeable handles should not be used while they are being recharged in the charger.
- The performance of the handles can only be guaranteed if genuine HEINE rechargeable batteries (NiMH, Li-ion L, see separate instruction leaflet for HEINE rechargeable batteries) and other spare parts are used.
- The bulb brightness can reduce during use if the rechargeable battery was not charged sufficiently before use. We recommend a minimum recharge time of 10 minutes for HEINE rechargeable batteries in HEINE chargers.
- In the interest of patient and general safety in use, the HEINE Standard F.O.LED / L LED incorporates two separate brightness levels:
 - If the battery is normally-charged, the LED provides a constant level of maximum brightness.
 - If the battery is discharged, the user is warned when the LED blinks or provides reduced brightness. In this case, the battery must be recharged or replaced.
- After some minutes use the laryngoscope bulb will be hot.
- The laryngoscope may only be used by trained personnel.

Application

HEINE Standard F.O. LED / L LED Laryngoscope rechargeable handles consist of four basic components:

- Handle shell (1)
- Bottom insert for 3,5V Li-ion L rechargeable battery (5)
- Insert (2)
- Bottom insert for 3,5V NiMH rechargeable battery (6)
- Li-ion L 3,5V rechargeable battery (3)
- Bottom insert for 2,5V replaceable batteries (8)
- NiMH 3,5V rechargeable battery (4)
- 2,5V replaceable batteries (7)

Changing the rechargeable battery:

- Unscrew the battery insert from the handle shell and remove it.
- Unscrew the bottom insert from the battery insert and remove it complete with the rechargeable battery.
- Pull the rechargeable battery out of the bottom insert.
- Insert the new rechargeable battery into the bottom insert with an audible „click“.
- Screw the bottom insert complete with the rechargeable battery back into the battery insert.
- Screw the battery insert back into the handle shell.

Changing the LED:

- The LED cannot be replaced by the user.

The rechargeable battery

This handle has been developed specifically for use with LED and for fast-charging the HEINE Li-ion L battery (see the instruction leaflet rechargeables batteries) HEINE in the NT 300 charger (for details see the instruction leaflet NT 300).

The NT 200 charger can also be used, but does not offer the same performance as it does not offer fast-charging, (a full charge can take up to 30 hours). The specification of the handle can be changed to other HEINE Standard F.O. Laryngoscope handles by using a different bottom cap, battery insert and rechargeable NiMH or dry batteries.

The bottom insert incorporates a charge status display which shows the state of charge of the battery. This activates automatically when the handle is removed from the charger or when it is switched on. After 10 seconds the display switches off.

The display gives the following information:

- Orange flashing: <10%
- Orange: 10–33%
- Orange/green: 33–66%
- Orange/green/green: 66–100%

Cleaning / Disinfection / Sterilising

The following instructions must be followed according to national standards, laws and guidelines. Any such measures must be carried out by qualified personnel.

For cleaning and disinfection of components, only those products which are approved for use on **medical devices with plated surfaces and plastics** (handles contain plastic components, e.g. bottom cap) may be used. After treatment, the instrument must be thoroughly rinsed with de-mineralised water to avoid reactions with following preparation steps and material damage. The manufacturer's recommendations for use must be followed.

Cleaning / Disinfection:

The handle shell can be cleaned and disinfected manually (wiping / soaking) or by machine (thermal disinfection up to 93 °C). Battery insert can be cleaned and disinfected manually (wiping) only.

Sterilisation:

Steam Sterilisation at 134 °C is allowed for the handle shell only.

Applicable standards:

DIN EN ISO 7376

(Anaesthetic and respiratory equipment – Laryngoscopes for tracheal intubation)

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Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries).

We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Any modification of a HEINE product with parts or additional parts which do not conform to the original HEINE specification will invalidate the warranty for the correct function of the product and further invalidate any warranty claims which result from such a change or modification. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded. For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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